For better dentistry

DENTSPLY

ceram .  [®] SphereTEC[™]
one
universal nano-ceramic restorative

Universal Nano-Ceramic Restorative

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ceram.x[®] SphereTEC[™] one

universal nano-ceramic restorative

Universal Nano-Ceramic Restorative

CAUTION: For dental use only.

USA: Rx only.

Content	Page
1 Product description.....	2
2 Safety notes	3
3 Step-by-step instructions	5
4 Hygiene.....	8
5 Lot number and expiration date.....	8

1 Product description

ceram.x[®] SphereTEC[™] one is a nano-ceramic, light-cured, radiopaque, universal composite based on the novel SphereTEC[™] filler technology indicated for both, direct and indirect restorations.

The spherical, granulated filler technology called SphereTEC[™] leads to a ball-bearing effect that takes the handling properties of ceram.x[®] SphereTEC[™] one to the next level:

- ceram.x[®] SphereTEC[™] one easily adapts to cavity surfaces but does not stick to hand instruments.
- ceram.x[®] SphereTEC[™] one is easy to sculpt precisely and has a high slump resistance.
- ceram.x[®] SphereTEC[™] one provides a firm consistency but is at the same time easy to extrude from syringes.

ceram.x[®] SphereTEC[™] one comprises of five universal CLOUD shades of intermediate translucency named A1 to A4.

Each CLOUD shade is designed to achieve shade match with a range of VITA[®] 1 shades. As a whole, the five CLOUD shades cover all 16 VITA[®] classic shades and allow for aesthetic clinical results.

The spherical, granulated fillers consisting of submicron primary glass fillers, provide a fast and easy polishing procedure resulting in an outstanding gloss.

1.1 Delivery forms

- Syringes

¹ VITA[®] is a registered trademark of VITA Zahnfabrik H. Rauter GmbH & Co. KG.

1.2 Composition

- Methacrylate modified polysiloxane (organically modified ceramic)
- Dimethacrylate resins
- Fluorescent pigment
- UV stabilizer
- Stabilizer
- Camphorquinone
- Ethyl-4(dimethylamino)benzoate
- Bis(4-methyl-phenyl)iodonium hexafluorophosphate
- Barium-aluminum-borosilicate glass
- Ytterbium fluoride
- Iron oxide pigments and titanium oxide pigments according to shade

Filler system based on SphereTEC™.

The filler system comprises barium glass, pre-polymerized filler and ytterbium fluoride (77-79 weight-% or 59-61 volume-%). Additional ingredients are catalysts, additives, stabilizers and pigments (~1 weight-%, according to shade). The total amount of inorganic fillers is 72-73 weight-% (48-50 volume-%). The particle size of the inorganic fillers ranges from 0.1 to 3.0 µm with a mean particle size of 0.6 µm.

1.3 Indications

- Direct restorations of all cavity classes in anterior and posterior teeth.
- Indirect fabrication of inlays, onlays and veneers.

1.4 Contraindications

- Use with patients who have a history of severe allergic reaction to methacrylate resins or any of the other components.

1.5 Compatible adhesives

The material is to be used in combination with a (meth)acrylate-based dentin/enamel adhesive such as the DENTSPLY adhesives Spectrum® bond, prime&bond® one ETCH&RINSE or prime&bond® one SELECT.

2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Warnings

The material contains methacrylates and polymerizable monomers which may be irritating to skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes, rinse with plenty of water and seek medical attention.

- **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and alcohol and wash thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use. Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- If refrigerated, allow material to reach room temperature prior to use.
- Use protective measures for the dental team and patients such as glasses and rubber dam in accordance with local best practice.
- The syringes cannot be reprocessed. To prevent syringes from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringes are handled offside the dental unit with clean/disinfected gloves. Discard syringes if contaminated.
- The shade guide and the individual shade guide tabs cannot be reprocessed. To prevent the shade guide and the individual shade guide tabs from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the shade guide and the individual shade guide tabs are handled with clean/disinfected gloves.
- Tightly close syringes immediately after use.
- Contact with saliva, blood and sulcus fluid during application may cause failure of the restoration. Use adequate isolation such as rubber dam.
- Interactions:
 - Do not use eugenol- or hydrogen peroxide-containing materials in conjunction with this product since they may interfere with hardening of the product.

2.3 Adverse reactions

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Irritation or possible allergic response. Reddish rashes may be seen on the skin.
- Contact with mucous membranes: Inflammation (see Warnings).

2.4 Storage conditions

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Store at temperatures between 2 °C and 28 °C (36 °F and 82 °F).
- Keep out of direct sunlight and protect from moisture.
- Do not freeze.
- Do not use after expiration date.

3 Step-by-step instructions

3.1 Shade selection

1. Prior to shade selection, clean the tooth surface with prophylaxis paste to remove plaque or surface stain.
2. Select the shade while teeth are hydrated.

For shade selection, there are two tools that may be used:

- The ceram.x® SphereTEC™ one shade guide. The ceram.x® SphereTEC™ one shade guide consists of tabs, which are manufactured from composite with optical properties close to those of ceram.x® SphereTEC™ one.
- The VITA® classical shade guide in combination with the i-shade label. The ceram.x® SphereTEC™ one i-shade label facilitates shade selection by assigning one of the five CLOUD shades to each of the 16 VITA® shades.

3.1.1 Shade selection with the ceram.x® SphereTEC™ one shade guide

1. Use the shade tabs of the ceram.x® SphereTEC™ one shade guide to select the CLOUD shade closest to the shade of the area to be restored.

Avoid contamination of the shade guide.

To prevent the shade guide from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the shade guide is handled offside the dental unit with clean/disinfected gloves. For shade selection, remove individual tabs from shade guide holder.

3.1.2 Shade selection with VITA® classical shade guide and i-shade label

Have the i-shade label ready.

To make sure that the i-shade label is available chairside, we recommend sticking it on the rear side of the VITA® classical shade guide.

1. Select the VITA® classical reference tooth, the central part of which best matches the area of the tooth to be restored.
2. Use the i-shade label (see table below) to identify the matching ceram.x® SphereTEC™ one CLOUD shade.

Tooth shade according to the VITA® Classical Shade Guide

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3.2 Cavity preparation

1. Clean the tooth surface with a fluoride free prophylaxis paste (e.g. Nupro[®] prophylaxis paste).
2. Prepare cavity (unless not necessary, e.g. cervical lesion).
3. Use adequate isolation such as rubber dam.
4. Rinse surface with water spray and carefully remove rinsing water. Do not desiccate the tooth structure.

3.2.1 Matrix placement and wedging (Class II restorations)

1. Place a matrix (e.g. AutoMatrix[®] or Palodent[®] V3 matrix system) and wedge. Burnishing of the matrix band will improve contact and contour. Pre-wedging or BITine[®] ring placement is recommended.

3.3 Pulp protection, tooth conditioning/dentin pre-treatment, adhesive application

Refer to adhesive manufacturer's Instructions for Use for pulp protection, tooth conditioning and/ or adhesive application. Once the surfaces have been properly treated, they must be kept uncontaminated. Proceed immediately to placement of the material.

3.4 Application

Apply material in increments immediately after the application of the adhesive. The material is designed to be cured in increments up to a 2 mm depth/thickness.

Avoid contamination of the syringe.

To prevent the syringe from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringe is handled offside the dental unit with clean/disinfected gloves.

1. Remove the cap.
2. Turn the handle of the syringe slowly in a clockwise direction and dispense the necessary amount of the material onto a mixing pad.
3. Point the front tip of the syringe upwards and turn the handle anti-clockwise to prevent oozing of the material.
4. Immediately close the syringe with the cap.
5. Protect the restorative material on the mixing pad against light.

3.4.1 Applying ceram.x[®] SphereTEC[™] one restorative

1. Fill the whole cavity incrementally. Each layer \leq 2 mm.
2. Cure each layer according to the table below [3.5 Light-curing].

3.5 Light-curing

1. Light-cure each increment with a suitable curing light². Cure each increment **for 10 seconds** when using a curing light as DENTSPLY SmartLite[®] PS or SmartLite[®] FOCUS[®] that has a minimum irradiance of 800 mW/cm².

Cure composite **for 20 seconds** when minimum irradiance is between 500 mW/cm² and 800 mW/cm².

Refer to curing light manufacturer's Instructions for Use for compatibility and curing recommendations.

² Curing light designed to cure materials containing camphorquinone (CQ) initiator. Peak of spectrum in the range of 440-480 nm.

Curing time for 2 mm increments

Shade	 mW/cm ²	 sec	 mm
A1 - A4	≥ 500	20	2
	≥ 800	10	

Curing time table.

The curing time table above also appears on all outer packages.

Insufficient curing.

Inadequate polymerization.

1. Check compatibility of curing light.
2. Check curing cycle.
3. Check minimum irradiance.
4. Cure each area of each increment for the recommended curing time.

3.6 Finishing and polishing

1. Contour the restoration using finishing burs or diamonds.
2. Use Enhance[®] finishing devices for additional finishing.
3. Prior to polishing, check that surface is smooth and free of defects.
4. For achieving a very high luster of the restoration, we recommend Enhance[®] and PoGo[®], finishing and polishing instruments, Prisma[®] Gloss[™] Regular and Extrafine polishing paste.

For contouring, finishing, and/or polishing, follow the manufacturer's Instructions for Use.

4 Hygiene

4.1 Syringes – cross-contamination

Cross-contamination.

Infection.

1. The syringes cannot be reprocessed.
2. To prevent the syringes from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringes are handled with clean/disinfected gloves. Do not reuse syringes if contaminated.
3. Dispose contaminated syringes in accordance with local regulations.

4.2 Shade guide and individual shade guide tabs – cross-contamination

Cross-contamination.

Infection.

1. The shade guide and the individual shade guide tabs cannot be reprocessed.
2. To prevent the shade guide and the individual shade guide tabs from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the shade guide and the individual shade guide tabs are handled with clean/disinfected gloves. Do not reuse shade guide and the individual shade guide tabs if contaminated.
3. Dispose contaminated shade guide and the individual shade guide tabs in accordance with local regulations.

5 LOT number () and expiration date ()

1. Do not use after expiration date. ISO standard is used: “YYYY-MM”
2. The following numbers should be quoted in all correspondence:
 - Reorder Number
 - Lot number
 - Expiration date

© DENTSPLY DETREY 2015-10-09

[These Instructions for Use are based on Master Version 01]

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DENTSPLY DeTREY GmbH
DENTSPLY Limited